TurnSheet

SystemRoMedic™

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Dansk

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Manual - Español

REF

2060

2080

2081

SWL: 300 kg/660 lbs

Table of contents

Always read the manual	3
English	4-5
Svenska	6-7
Norsk	8-9
Dansk	10-11
Deutsch	12-13
Netherlands	14-15
Français	16-17
Italiano	18-19
Español	20-21
<hr/>	
Combination	22
Size and Accessories	22

**EN Always read the manual**

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TurnSheet is bedded in the bed and features an anti-friction material that makes turning or repositioning higher up in bed easier for both the patient and the caregiver. TurnSheet consists of two sections: a mattress cover and a turning cover. The mattress cover is bedded around the mattress and the turning cover is then placed over it. The mattress cover has a mid-section with anti-friction material. The sides have friction material that prevents sliding.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

Functional inspection


Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Functional inspection

Test TurnSheet's sliding function by rolling it and noting how easily it glides. If possible, compare with a new product.

Similarly, test the friction material in the same way. Check to ensure that the surface is rough and still provides friction.

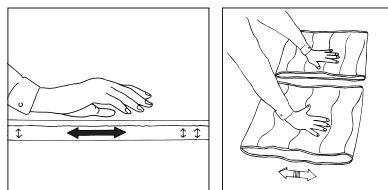
Note


Be aware that there is a risk of the patient sliding out of bed. Take care to ensure that the mattress is sufficiently wide, so that there is anti-slip material on both edges of the mattress cover.

Placement

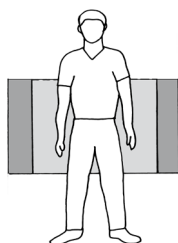
The mattress cover is placed with the sliding surface in the middle of the bed and the rough friction surfaces evenly distributed on both sides. The mattress cover is secured around the mattress with Velcro strips.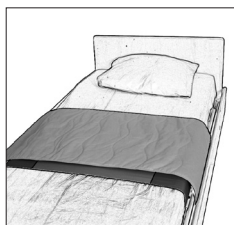


The turning cover is placed over the mattress cover. A draw sheet can be placed on top of the turning cover.

Using TurnSheet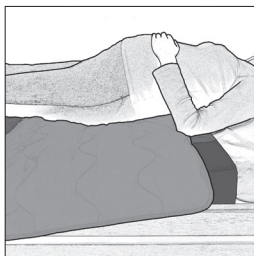

The patient can use TurnSheet himself/herself if he/she is able to turn with a little help from a caregiver.

Since the TurnSheet glides easily, it makes turning easier for the caregiver. Remember to work in walking stance with knees bent, arms straight, and shift the weight.

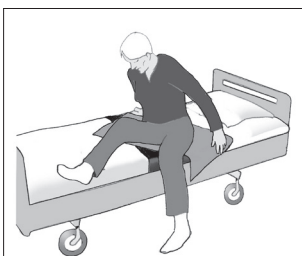TurnSheet can also be used to help the patient to rise to a seated position at the edge of the bed. The patient can rotate easily to one side and lift the feet over the edge of the bed.

Material:

Polyamid, polyester, microfibre

Care of the product:

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

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SystemRoMedic™

For 25 years we have applied ourselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Our mission, to help people, has always been, and will continue to be, the driving force of innovation. We love easy transfers.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

handicare

Handicare AB
Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com