REF  
7030

**Max: 120 kg/264 lbs**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>4</b>
<b>Norsk</b> .....	<b>5</b>
<b>Dansk</b> .....	<b>6</b>
<b>Suomi</b> .....	<b>7</b>
<b>Deutsch</b> .....	<b>8</b>
<b>Nederlands</b> .....	<b>9</b>
<b>Français</b> .....	<b>10</b>
<b>Italiano</b> .....	<b>11</b>
<b>Español</b> .....	<b>12</b>

---

<b>Product Information</b> .....	<b>23</b>
----------------------------------	-----------

<b>Accessories</b> .....	<b>23</b>
--------------------------	-----------

MobiStol is mounted on a chair to make it mobile. The chair can now be used for shorter transfers indoors, such as in and out from a table.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

## Always read the user manual

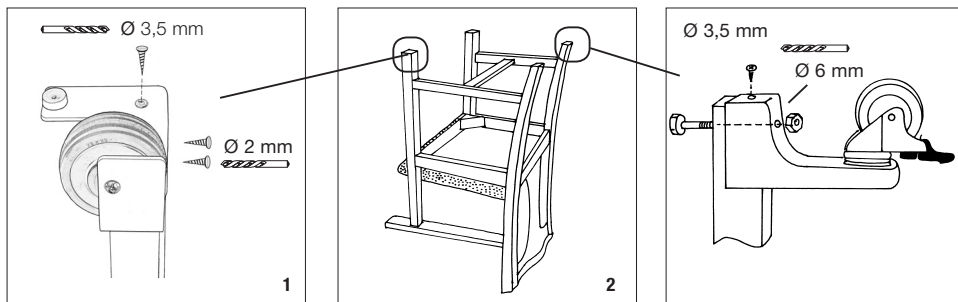
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Assembly instructions

The wheels fit most chairs with square or rectangular legs made of a hardwood, e.g. beech, oak or birch, measuring 3x3 cm or larger.

Turn the chair upside down and hold the metal mounts with wheels against the chair legs, as shown in pictures 1 and 2.

Mark the position of the screw holes.

Drill about 2 cm deep for the woodscrews and straight through the rear legs for the bolts.


**NOTE!** The front wheels should be mounted on the outside of the legs, while the metal mounts for the rear wheels should be mounted on the inside and back.

**NOTE!** Drill the holes for the bolts from back to front.


**NOTE!** Check the screws and tightened them again after a few days.

## Mounting of wheels on rear legs with different angles

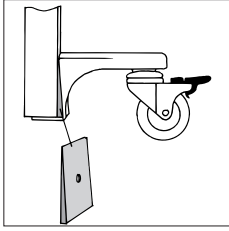
---

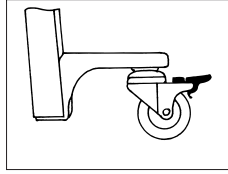If the chair's rear legs are not inclined, plastic wedges are used.

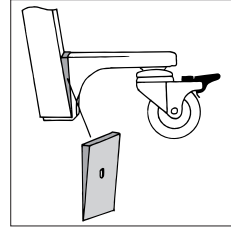
If the chair's rear legs are slightly inclined, no plastic wedges are used.

If the chair's rear legs are steeply inclined, plastic wedges are used.

## Transfers

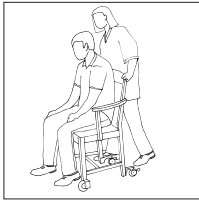
---

Always hold the chair and check to ensure that the wheels are locked before the patient sits in the chair or stands up from the chair. The wheels must be locked while the patient is left in the chair.

## Locking rear wheels

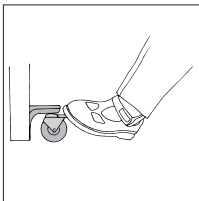---

Lock the rear wheels by pressing down on the small metal plate with your foot. Disengage the wheel brake by lifting up on the plate with your foot.

## Care of the product

Clean the wheels regularly to keep them free of dust, dirt and hair.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Vida útil estimada

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## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

**handicare**

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)