RisePorto450 RisePorto300

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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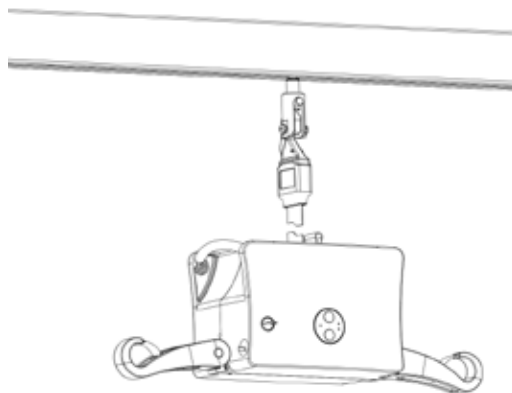
REF


50200015

50200016

**SWL: 140 kg/300 lbs
205 kg /450 lbs**

Table of contents

English	4
Svenska	18
Norsk	32
Dansk	46
Suomi	60
Deutsch	74
Nederlands	88
Français	102
Italiano	116
Español	130

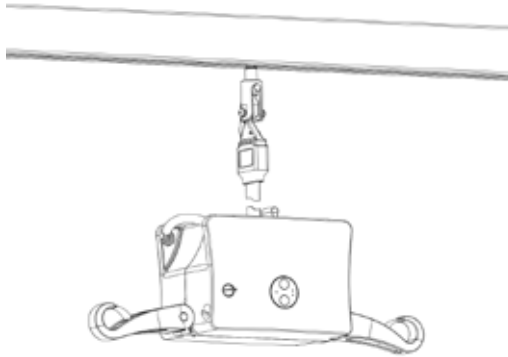RisePorto is a portable ceiling lift unit featuring unusually small measurements and extremely low weight. At the same time, it is both strong and safe. When used with the appropriate lifting accessories, RisePorto provides secure, comfortable and individually adapted lifting of users in connection with seated or horizontal transfers to and from the bed, wheelchair or toilet. RisePorto offers a large lifting range and the possibility of both very low and very high lifting, making it an excellent solution also for lifting from the floor and for gait training and standing training. RisePorto can also be used for convenient transfer of users from one room to another.

Functional inspection

Visual inspection

Inspect lift functions regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted in the trolley.

Check rail end stops.

Check safety latch function.

Check lifting movement.

Inspect the lift for signs of wear and damage.

Check the hand control cable for signs of wear.

Check to ensure that hand control markings coincide with lift functions.

Check battery charge level.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Under no circumstances may the lift be used by persons who have not received instruction in the operation of the lift.

It is strongly prohibited to modify the original product.

Mounting

Unpacking and check

The RisePorto package includes the following:

- 1 RisePorto lift unit
- 1 hand control
- 1 user manual
- 1 charger
- 1 key for manual emergency lowering

Check to ensure that no parts have been left in the packaging.

Check to ensure that the lift has no signs of damage from transportation.

NOTE!

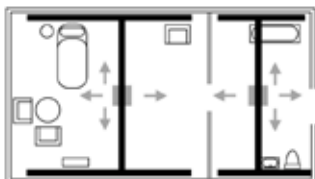
The lift unit and the trolley must be installed and first-time inspected by authorized personnel (inspector) in accordance with the installation instructions provided by Handicare AB. Periodic inspection of the equipment should be undertaken at least once a year. Original spare parts should be used and service/maintenance of the equipment should be in accordance with the user manual.

The lift unit must be charged for at least 3 hours before first- time use.


Different rail systems

The choice of rail system is determined by the lifting needs at hand.


Rail systems with full room coverage, in bedroom and bathroom

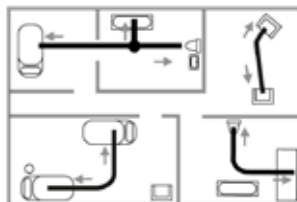Basic rail systems, with curves and turn table

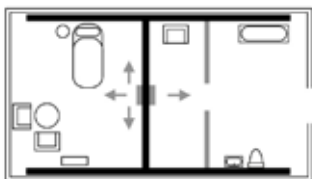Rail system with full room coverage, above bathroom wall


Freestanding lift stand


Mounting

The lift unit is easier to mount in the trolley if the trolley is placed against an end stop on the rail. First release enough lift strap by pulling lightly on the hook or the lifting strap while at the same time pushing the Down-button on the hand control or on the control panel on the lifting unit.

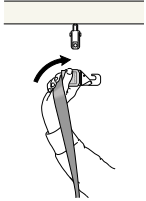
Mount the hook as shown below


Turn the hook as shown

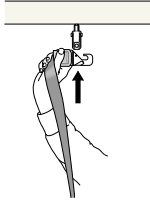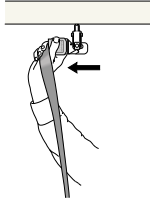
Hold the hook as shown on the image while at the same time moving it towards the rail.

Insert the hook in to the trolley.

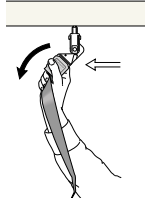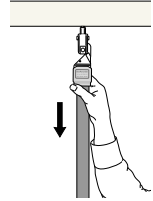
Turn the hook downwards while at the same time pushing it towards the trolley.


Let go of the hook and pull lightly in the lifting strap to ensure it is secure in the trolley.

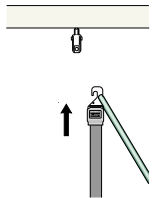
Mount the hook with the handle for mounting (Handle for mounting, 50200014)

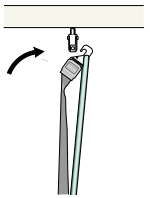
Mount the handle for mounting on the hook as shown.

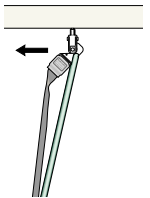
Hold the hook as shown on the image while at the same time moving it towards the rail.

Insert the hook into the trolley.

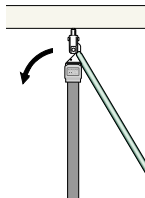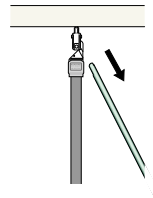Turn the hook downwards while at the same time pushing it towards the trolley.

Straighten the strap and pull the handle for mounting loose. Make sure that the hook is secure in the trolley. Note! The handle for mounting must be removed!

Make sure that the handle for mounting is removed.
Turn the lifting unit and make sure that the hook is also turning.

NOTE! The lift unit must hang freely.

Final inspection

- The lift unit and the trolley must be installed and first-time inspected by authorized personnel (inspector) in accordance with the installation instructions provided by Handicare AB. Periodic inspection of the equipment should be undertaken at least once a year. Original spare parts should be used and service/maintenance of the equipment should be in accordance with the user manual.
- Check to ensure that no parts have been left in the packaging.
- Check to ensure that the lift has no signs of damage from transportation.
- Check emergency stop function by pressing the emergency stop, and then pressing either the up or the down button. If nothing happens when the up or down buttons are pressed, the emergency stop is functioning properly.
- Check the entire length of the strap for signs of damage and inspect seams for wear. Press the up button and run the strap all the way up. Then, press the down button and run the strap all the way down.
- Test lift function by lifting a person (not a user) using an approved lifting sling. At the same time, check both the user manual and electrical emergency lowering function with someone in the lift. See section on Emergency lowering.
- Check to ensure that the rail system is equipped with end stops.
- Check the hand control cable for signs of wear.
- Check to ensure that hand control markings coincide with lift functions.
- The user/attendant shall always pay attention to and report any unusual noises or vibration of the equipment.

NOTE!

Before the lift is used for the first time it must be charged for at least 3 hours. See section on charging the lift unit.

RisePorto is delivered with the emergency stop activated to ensure that the battery does not drain during transportation and storage. Activate the lift unit by pulling the emergency stop outwards until a click sound is heard.

Keep the user manual where it is accessible to users of the product.

Always keep the key for mechanical emergency lowering close by the lift unit. Everybody who uses the lift unit shall be informed where the key is kept.

Using the product

Important information

- The RisePorto lift unit can be moved between different rail systems by the attendant. The lift unit must only be used in combination with the Trolley 360 RisePorto.
- The attendant shall be trained professional with thorough understanding of the equipment and lifting of disabled persons. It is important that proper accessories and techniques are used by the attendant and it is also very important that a user never is left unattended in a lifting situation.
- It is strongly prohibited to modify the original RisePorto product.
- Warranty does not apply if repairs or alterations are made to the lift by personnel who are not authorized.
- RisePorto must not come in direct contact with water.
- The lift must not be charged in a wet room.
- To ensure optimal function, the lift must be inspected regularly. See section on Maintenance.
- Under no circumstances may safe working load be exceeded. See section on technical information and the product label on the lift.
- Lifting accessories must be properly fitted and tested in relation to the user's needs and functional ability.
- articular care must be taken when using the equipment near strong sources of interference, such as diathermy apparatus, so that e.g., diathermy cables and similar equipment are not placed near the device. When in doubt, consult your equipment technician or the supplier.

Before use

Ensure that the product is correctly mounted in the trolley.

Check rail end stops.

Check safety latch function.

Check lifting movement.

Inspect the lift for signs of wear and damage.

Check the hand control cable for signs of wear.

Check to ensure that hand control markings coincide with lift functions.

Check battery charge level.

Safe working load

Different products on the same lift system (lift unit, lifting sling and other lifting accessories) may have different safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Description of functions and symbols

RisePorto has many different built-in functions and each function is explained in the related section of this user manual. Below is a short description of the available functions.

- Electronic soft start and stop
- Electric and manual emergency lowering
- Emergency stop button on the lift cover
- Over speed protection
- Overload protection
- LED symbol showing when service is needed
- LED symbol showing battery status and if the lift unit is charging
- Integrated handlebars
- Integrated foldable sling bar
- Room transfer loop

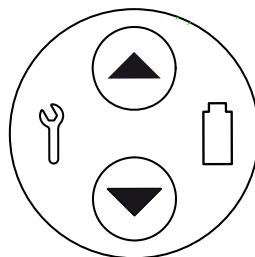
Symbols

Symbol for battery status.


Symbol showing when the lift unit needs service.

Charging the lift unit

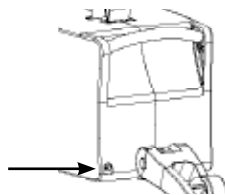The lift unit is charged using a stand-alone charging unit (Mascot2215, 81000012) that is connected to the lift unit via a DC plug. The charger is placed in an appropriate place. Remember that the charger is separate and easily can be mixed up with other chargers. Make sure to use the correct charger, see Technical information.

Battery status

The battery symbol is lit green when more than 50% of the capacity is left.

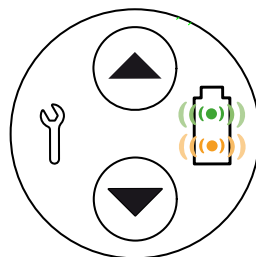
The battery symbol flashes green when 25-50 % of the capacity is left.

If the battery symbol flashes green and orange at start up, the battery needs charging.

If the battery symbol is dark and an audio signal is heard when lifting upwards the battery is drained and should be charged immediately.

When the charger is correctly plugged into the lift unit the battery symbol is constantly lit green.

When the lift unit is activated the battery symbol lights up for 2 seconds to show the current battery status.

NOTE!

Use only a charger intended for the lift unit.

Charge the battery regularly for maximum life time. Do not allow battery charge to fall below 25% (green and orange battery symbol flashes on the lift units information display).

The emergency stop must not be activated when charging.

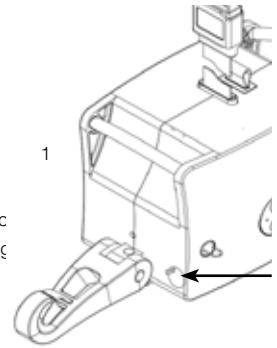
Replacement of the battery may only be done by authorized personnel.

Hand control


Connection of hand control to lift unit

The outlet for the hand control (1) is on the short side of the lift. Connect the hand control firmly.

Raising/lowering

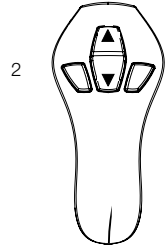The buttons with the black arrows (2) raise and lower the lift unit. Motion stops as soon as the button is released. The lift unit can also be activated with the buttons for raising/lowering.

On/off via hand control

The hand control is equipped with a green button (ON) to activate the lift unit and a button (OFF) to shut it off (3). The lift unit automatically shuts off after 15 minutes of inactivity. Normally, the lift can be activated via the green (ON) button before use.

Activate/start the lift unit

The lift unit is activated by pressing the green button (ON) or the Up- or Down-button on the hand control. When this is done the battery symbol on the lift unit's information display is lit for 2 seconds, showing the current battery status. If the battery symbol does not light and the lift unit is not activated, check that the emergency stop is not activated (pressed in).

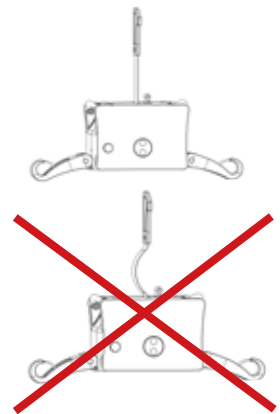
Active safety

RisePorto has a built-in slack strap switch requiring that, in order to be manoeuvred up or down, RisePorto must be loaded with at least 2 kg or its own weight. This is a safety feature that prevents a slack lift strap from being wound in or out from the lift unit causing a short fall when next in use.

RisePorto automatically stops if the lift strap is unloaded, for example if it is lowered onto a table. If this happens, the lift strap must be manually stretched in order to be manoeuvred up or down.

If the lift strap is twisted when lifting upwards, the lift unit automatically stops and can only be manoeuvred down.


In order to save battery the lift unit automatically turns off after 15 minutes of inactivity.

Information display


The information display on the long side of the lift indicates battery capacity (1) and if service is needed (2). The battery status is shown for 2 seconds upon activation (ON) of the lift unit.

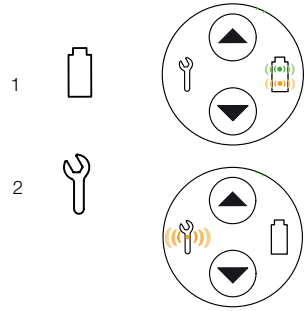More information on the respective function can be found under Preventive maintenance and Charging the lift unit.

Preventive maintenance

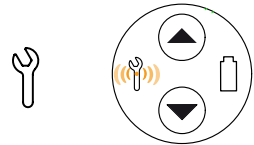
When the lift unit has been used for 12 months the service indicator on the information display will light up and stay lit as long as the lift unit is activated. Contact an authorized service technician for maintenance and resetting.


Emergency stop

To activate the emergency stop: Push the red emergency stop button on the long side of the lift unit.

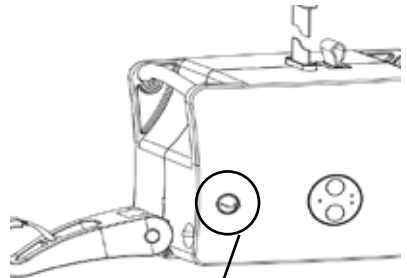Resetting: Pull the emergency stop button until an audio signal is heard.

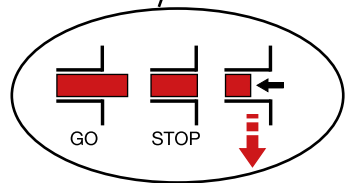NOTE! If the emergency stop button is held by hand at its most inner position the electrical emergency lowering will be activated. As soon as the button is released the lift unit returns to emergency stop mode.

Electrical emergency lowering

If the hand control does not function, electrical emergency lowering can be done in two ways: Alternative 1, by pressing the down arrow on the lift units information display. Alternative 2, by pressing the emergency stop button to its most inner position and holding it there until the user is at an appropriate height.

 Emergency lowering should take place only if no other electrical function works.

When using the electrical emergency lowering special attention must be taken to the lift's motion. Lower the user down to safe height and not any further.

Resetting: The emergency stop button automatically goes back to emergency stop mode as soon as it is released. After evaluation of the situation the emergency stop may be deactivated by pulling the button outwards until an audio signal is heard.

Always ensure that emergency lowering takes place in an appropriate location.

Manual emergency lowering/raising

Place the emergency lowering key supplied with the lift unit into the hole. Turn the key counterclockwise for lowering until the user is at an appropriate height.

NOTE!

The manual emergency lowering requires numerous rotations.

Trouble-shooting

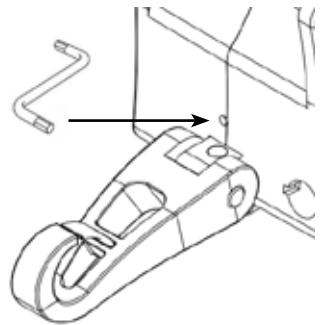
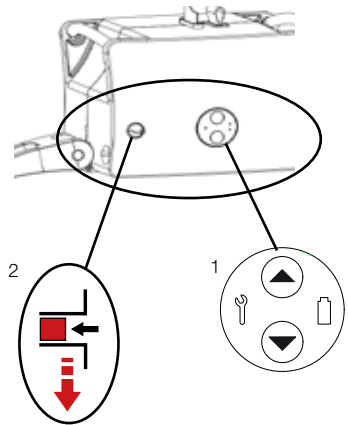
If the lift cannot be activated, check that:

- the emergency stop button is not pushed in.
- all cables are properly and securely connected.
- battery charging is not in progress.
- the battery is charged.
- the lift strap is loaded.

If the lift unit is not working properly, contact your dealer.

If the lift makes unusual noises:

- Try to determine the source of the sound. Take the lift out of operation and contact your dealer.

Accessories

Item No.	Product	Description
50200014	Trolley 360 RisePorto	360° turning trolley
50200004	Room transfer strap	300 mm / 11.8"
70200021	Room transfer strap	700 mm / 27.6"
70200022	Room transfer strap	800 mm / 31.5"
70200023	Room transfer strap	900 mm / 35.4"
70200024	Room transfer strap	1000 mm / 39.4"
50200017	Handle for mounting	750 mm / 29.5"
70200053	Lift strap extension	370 mm / 14.6"
70200054	Lift strap extension	570 mm / 22.4"
70200055	Lift strap extension	670 mm / 26.4"
70200056	Lift strap extension	770 mm / 30.3"
70200057	Lift strap extension	870 mm / 34.2"
70200058	Lift strap extension	970 mm / 38.2"
70200048	Software for diagnostics	Software and cable

Lifting slings

Handicare's SystemRoMedic™ line includes a wide range of functional and comfortable, high-quality lifting slings that are adapted for all types of lifting and for users with different needs. The lifting slings are available in several materials and in sizes ranging from XXS to XXL. There are also special lifting sling models in XXXL and XXXXL for extremely large and heavy users. All models are safe and very easy to use.

Maintenance

The lift must undergo thorough inspection at least once per year. Inspection must be performed by authorized personnel and in accordance with Handicare's service manual.

Repairs and maintenance may only be done by authorized personnel using original spare parts.

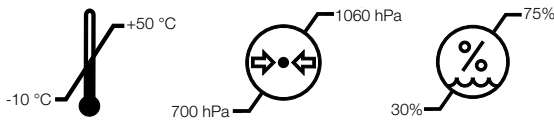
Used batteries are to be left at the nearest recycling station. Used batteries can also be returned to Handicare or a Handicare dealer for recycling.

Cleaning/disinfection

If necessary, clean the lift with warm water or a soap solution and check that the castors are free from dirt and hair. Do not use cleaning agents containing phenol or chlorine, as this could damage the aluminium and the plastic materials. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

Storage and transportation

If the lift is not to be used for some time or e.g., during transport, we recommend that the emergency stop button be pressed in. The lift should be transported and stored in $-10\text{ }^{\circ}\text{C}$ to $+50\text{ }^{\circ}\text{C}$ and in normal humidity, 30% -75 %. The air pressure should be between 700 and 1060 hPa. Let the lift reach room temperature before the batteries are charged or the lift is used.

Service agreements

Handicare offers service agreements for maintenance and regular testing of your lift unit. Contact your local Handicare representative.

Symbols

	Read user manual
	May not be discarded in domestic waste
	The product complies with the requirements of the Medical Devices Directive 93/42/EEC.
	Type B, according to the degree of protection against electric shock.
	The device is intended for indoor use.
	Class II equipment
	Always read the user manual

EMC

This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as reorienting or relocating the equipment/system or shielding the location.

Serial number



Technical Information

Lift motor:	24VDC
Lifting speed:	30 mm/s (1.2 inch/s) without load
Charger IN:	Mascot 2215 100-240VAC/ 50-60Hz Max 0.4A, art. no:81000012
Charger OUT:	41 VDC $\pm 0,3$ Max 0.4A
Batteries	26.4 VDC (22 pcs. 1.2 VDC) 2.7 Ah. NiMH22XAA2500 HR-3U-2500
Motor cover:	Non flammable ABS plastic
Hand control:	Electrical, 4 buttons
Emergency lowering:	Manual and electrical
Lift weight	5.8-6.0 kg, 12.8-13.2 lbs
Lifting range:	1700 mm, 66.9 inch
SWL RisePorto450:	205 kg, 450 lbs
SWL RisePorto300:	140 kg, 300 lbs
Sound level:	Unload: 55 dB, Max load: 57 dB
Pressure hand control:	4 N
Lift measurements:	288 x 152 x 153 mm, 11.3 x 6.0 x 6.0 inches
Intermittent operation:	Op 10/90, active op. max. 2 min. Out of a time of 100, active time must be less than 10, though not more than 2 min.
IP class, lift unit:	IP X3
IP class, hand control:	IP X4
Expected lifetime:	10 years

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

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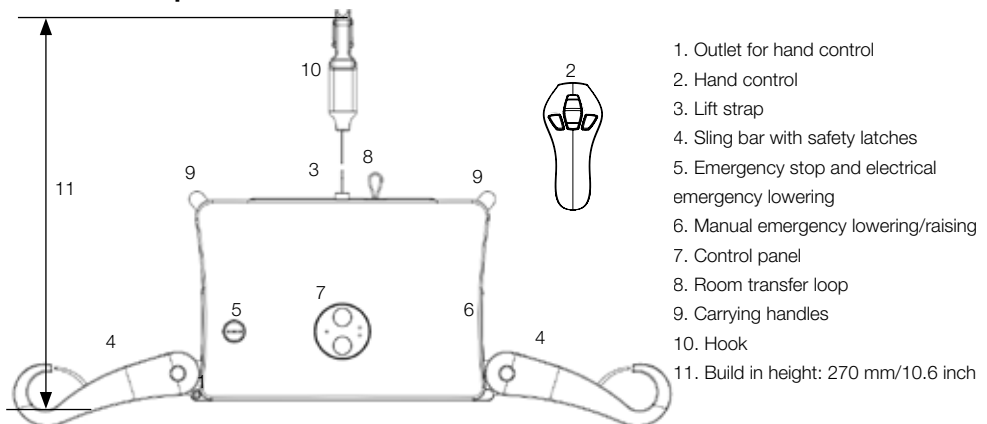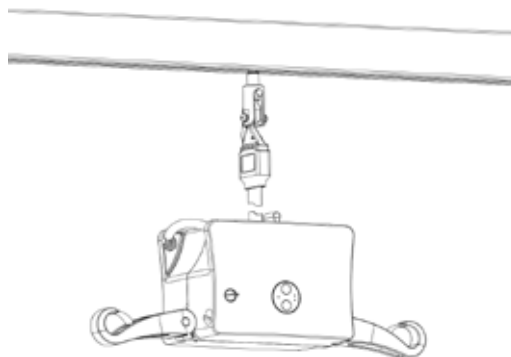Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Maskinvägen 17
SE-972 54 Luleå , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com